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Hydrogen Peroxide Sterilizer

工作原理 Work Principle

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SV series Hydrogen peroxide sterilizer applies low concentration hydrogen peroxide fogging combined with high frequency and high pressure ionization technology to carry on room sterilization. Hydrogen peroxide produces free OH⁻ during the process, which causes oxidative damage to cell components, thus achieving the purpose of eliminating bacteria.

性能特点 Performance Characteristics

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1. Use high frequency and high voltage ionization technology

Improve the permeation and diffusion ability of the effective component of hydrogen peroxide disinfectant, solve the permeation of small space, no dead angle

2. Spray in all direction

360° automatic horizontal rotation of the spray head, combined with 60° upward adjustment of the nozzle, to achieve a full range of spray

3. Safety low concentration disinfectant

7.5% Hydrogen peroxide disinfectant belongs to non-hazardous chemicals, and its purchase and use of no need for the public security department record

4. High effictive disinfectant

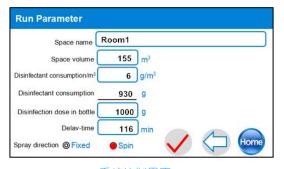
Hydrogen peroxide is a type of high effective disinfectant, which has the characteristics of broad spectrum, high efficiency, non-toxic, etc., and the antibacterial ability is 6-log

5. Quaick connect disinfectant container

Special design of the disinfectant container cover and one end of the nozzle pipeline fast insertion structure, easy to achieve the connection of disinfectant

6. Built-in air pump

Low noise and high quality internal air pump with corrosion resistance treatment, no external air source is required

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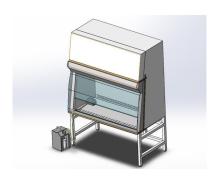
A type of strong oxidizing and efficient disinfectant

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Non-toxic, easy to decompose, disinfection and sterilization can be quickly decomposed into ${\rm H}_2{\rm 0}$ and ${\rm O}_2$

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Good material compatibility, high concentration of hydrogen peroxide will bring some corrosion

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The concentration of Hydrogen peroxide solution higher than 8% are hazardous chemicals that are prone to explosive production, and their purchase and use are subject to strict record keeping control by public security departments

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